


Foam Hip Abduction Orthosis

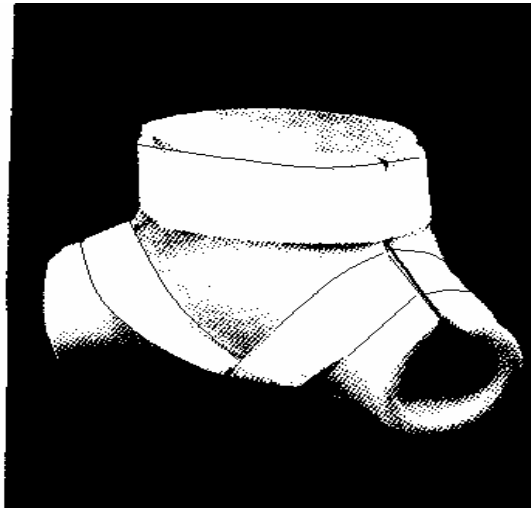

Care and Use Instructions

1. This orthosis is designed to hold the child's legs in a spread position. This position helps hold the hip in the socket. It can be used in cases of hip dysplasia or congenital hip dislocation.
2. The orthosis is to be worn over the diaper.
3. The orthosis is donned (put on) as if you were putting on a pair of shorts. Open the velcro straps and place the child down in the the orthosis, with the legs in the spread position. Then have the child lie back and secure the velcro straps. The particular order does not matter.
4. The orthosis should be worn at all times unless specified by the attending physician.
5. The orthosis can easily be washed with soap and water, making sure it is dry before placing it back on the child.
6. If you have any questions or problems, please do not hesitate to call your physician or MOPC @ 800 598 6672