


## **Hip Abduction Orthosis**

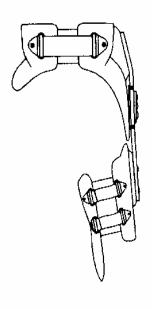

## **Care and Use Instructions**

- 1. This orthosis is to be worn at all times to protect the hip from dislocating.
- 2. It is possible to briefly remove the orthosis for bathing purposes. Care must be taken when moving as to not put stress on the affected hip.
- 3. The hip joint will allow flexion of the hip to 90 degrees but it will stop it from going into any extension. The hip is also abducted about 25 to 30 degrees.
- 4. It is possible to launder the pads inside the orthosis. Just replace the pads with some towels while the pads are drying.
- 5. Do expect to see some pinkness on the inside of the knee, as this is a pressure point of the brace to hold the leg in abduction. Pink is okay, just not the reds and purples.
- 6. If you have any questions about the brace please call Morgantown O & P Center at 1 (800) 598-6672.